OMNI MAX

QUICK START GUIDE

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1 Introduction

This guide will help you get started using the OMNI MAX Scanner to identify pets. For further information, please ensure your reader is powered down and connect it to a computer to access the complete user manual located within the "doc" folder of the reader's memory.

To conveniently interface OMNI MAX to your computer, or to check for newer versions of the OMNI MAX firmware, we recommend using S-ID companion software, freely downloadable from www.datamars.com

Microchip technologies supported

- FDX-B, FDX-A/Fecava, Trovan, Avid Encrypted ("OMNI MAX" standard version)
- FDX-B, FDX-A/Fecava, Trovan, HDX ("OMNI MAX i" version)

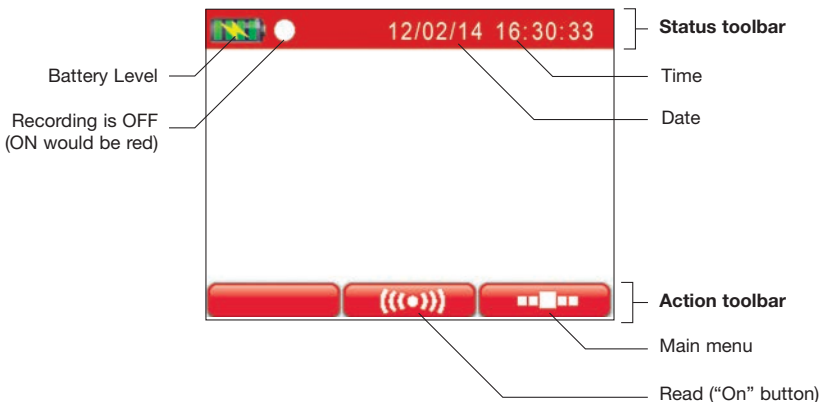
How to correctly hold OMNI MAX

2 OMNI MAX Basics

Switching On the Scanner

- Press the "ON" button which is located in the center of the scroll-wheel for two seconds
- The Home screen will appear:

Switching Off The Scanner

- OMNI MAX has an “Auto-off” function that automatically turns off the reader after 2 minutes (default value) of inactivity. To turn off the scanner manually, proceed as follow:

- Press the right navigation button  located on the Home menu to access the Main menu
- Once the Main menu appears, press the left navigation button that points to the following icon:

Accessing the Main and Advanced menus

- Once the reader is switched on, press the right navigation button to access the Main menu.  Rotate the scroll-wheel to access main menus and settings:

- Once in the Main menu, select the Advanced Settings icon.  Rotate the scroll-wheel to access the different Advanced menus and settings.

Single reading / Continuous reading

Memory File enabled (recording) / Memory File disabled (no recording)

USB as Key Board Wedge / USB as Virtual Com Port

Microchip code displayed on 15 digits / displayed on 23 digits

Access to memory files folder

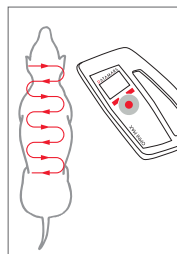
Bluetooth off / Bluetooth as KeyBoard Wedge / Bluetooth as Virtual Com Port
*** Bluetooth available as optional*

3 Using OMNI MAX to Scan Microchips

Scanning a Microchip

- Once OMNI MAX is turned ON, hold it in close proximity to the animal, and then press the center of the scroll-wheel
- OMNI MAX will emit an audible “beep” when the scanner successfully detects a microchip
- To stop scanning if no microchip has been found, press the center of the scroll-wheel

Effective Scanning

- Hold the scanner close to, or touching the animal
- Scan slowly and repeatedly over the entire body to check whether or not a microchip is present
- Wave the scanner back and forth lightly while scanning, because microchips can be in various orientations within the animal
- Scan first in a horizontal, then a vertical “S” pattern down the animal’s body, as shown in the diagram beside
- Consider scanning each animal more than once to ensure any present microchip can be localized by the scanner

4 Charging the Battery

OMNI MAX can be charged in two ways. You can connect the USB cable to the wall-mounted adapter or you can connect the USB cable to a USB port on your PC.

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