

# COMPACT MAX

## QUICK START GUIDE

V 2.0



# DATAMARS

Americas Headquarters:  
250 West Cummings Park  
Woburn, MA 01801, USA  
Phone: +1 781 281 2216  
Fax: +1 781 300 7330

[animal-id@datamars.com](mailto:animal-id@datamars.com)

[www.pet.datamars.com](http://www.pet.datamars.com)


© 2017 DATAMARS

# 1 Before you start

We recommend to fully charge the battery before first usage. COMPACT MAX can be charged either by connecting the USB cable to the wall-mounted adapter or to a USB port on your PC.

## Microchip technologies supported

- FDX-B, FDX-A/Fecava, Avid Encrypted, Trovan (“COMPACT MAX” US version)

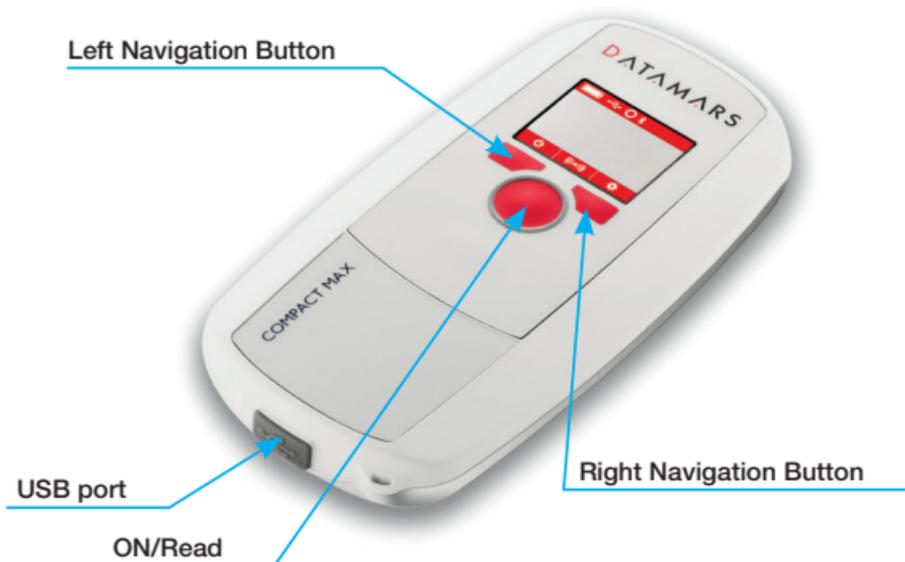


To conveniently interface COMPACT MAX to your computer, to check for newer versions of the COMPACT MAX firmware or to modify the default settings, we recommend using S-ID companion software, freely downloadable from [www.pet.datamars.com](http://www.pet.datamars.com)

# 2 COMPACT MAX Basics


## Switching On the Scanner

- Press the “ON/READ” button.
- The Home screen will appear:

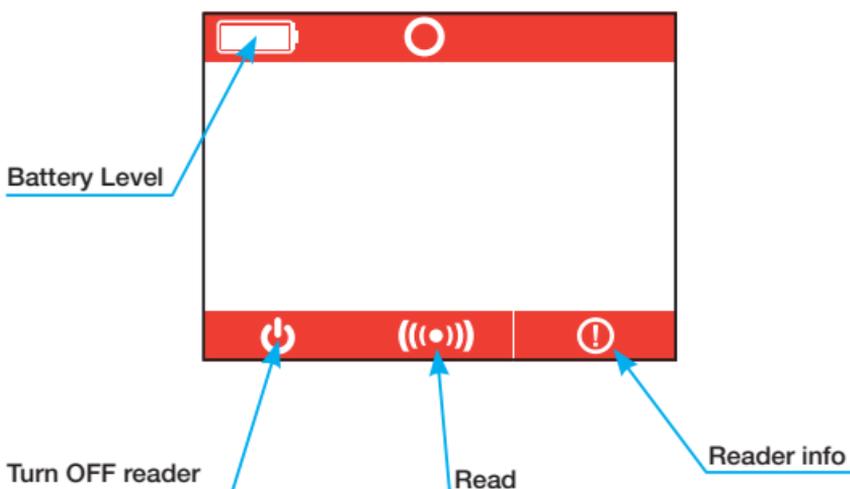


### 3 Using COMPACT MAX to Scan Microchips

- From the home menu, press the "ON/READ" button to start reading

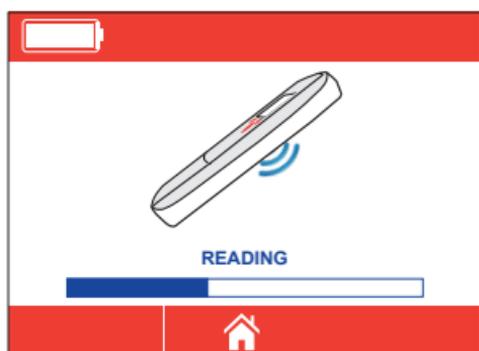

- Hold the scanner close to, or touching the animal
- For optimal reading, keep the scanner parallel to the animal's body rather than pointing it at the animal
- Scan slowly and repeatedly over the entire body
- Wave the scanner back and forth slowly while scanning. Microchips can be in various orientations
- Scan first in a horizontal, then a vertical "S" pattern down the animal's body, as shown in the diagram below
- Metal can interfere with microchip signals. Try to scan away from metal exam tables, metal doors, metal desks... (collar tags or leash connectors are not a problem). Scanning through a metal or metal mesh cage will probably interfere with reading

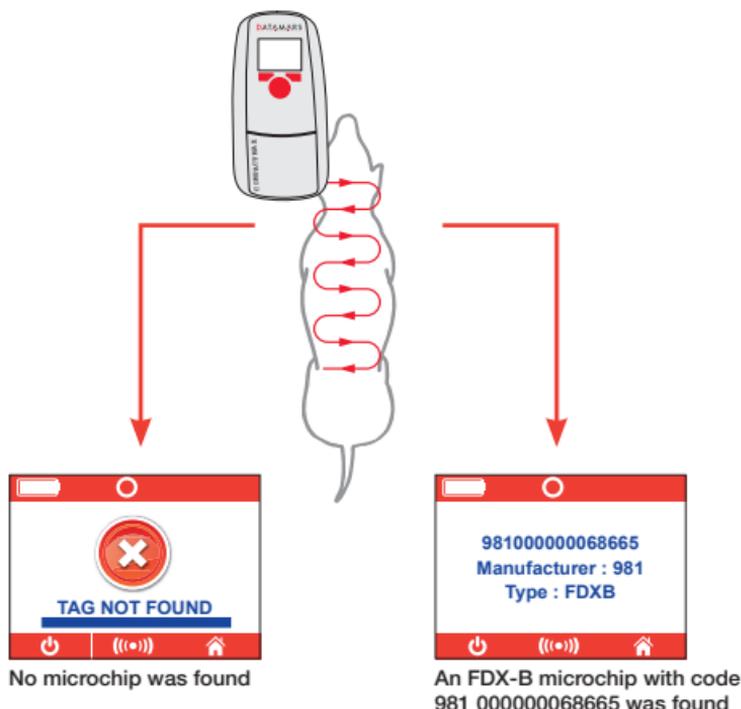

- Consider scanning each animal twice if no microchip has been found at first scan.

# WARRANTY CARD

The warranty covers damage caused by faulty material or manufacturing defects for the period of 1 year from the date of purchase.

Equipment repaired by a third party, as well as damage caused by the customer are not covered by this warranty. Claims for covering of damage exceeding the limits of the present warranty are not accepted.

In case of repair or replacement the warranty is neither extended nor renewed.

In the event of a claim under this warranty contact your dealer directly and present the warranty certificate stamped and dated by the dealer or the purchase invoice. Repairs carried out without a valid warranty certificate will be invoiced to the customer.

Model:

Serial number:

Date :

Distributor:

# DATAMARS

Americas Headquarters:  
250 West Cummings Park  
Woburn, MA 01801, USA  
Phone: +1 781 281 2216  
Fax: +1 781 300 7330

[animal-id@datamars.com](mailto:animal-id@datamars.com)

[www.pet.datamars.com](http://www.pet.datamars.com)

0215