COMPACT MAX

QUICK START GUIDE

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GUIDA RAPIDA

V 2.0

DATAMARS

Corporate Headquarters:
Via ai Prati
6930 Bedano-Lugano
Switzerland
Phone: +41 91 935 73 80
Fax: +41 91 945 03 30

animal-id@datamars.com

www.pet.datamars.com

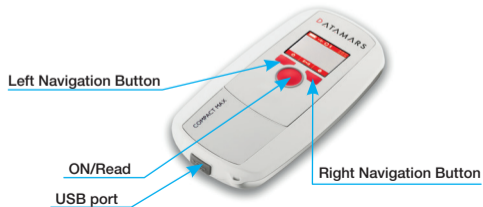
1 Before you start

We recommend to fully charge the battery before first usage. As the device may discharge during storage, please make sure that the battery is charged regularly. COMPACT MAX can be charged either by connecting the USB cable to the wall-mounted adapter or to a USB port on your PC.

To conveniently interface the scanner to your computer, to check for newer versions of the firmware or to modify the default settings, we recommend using S-ID companion sw, freely downloadable from www.pet.datamars.com

Microchip technologies supported

- FDX-B, FDX-A/Fecava


Icons Legend of the Settings menu

-  Calendar setup (date, time and format)
-  Memory File enabled (recording) /  Memory File disabled (no recording)
-  USB as Key Board Wedge /  USB as Virtual Com Port
-  15 DIGITS /  23 DIGITS Microchip code displayed on 15 digits / displayed on 23 digits
-  Delete memory content
-  Info (Firmware and bootloader versions)

2 Screen Navigation

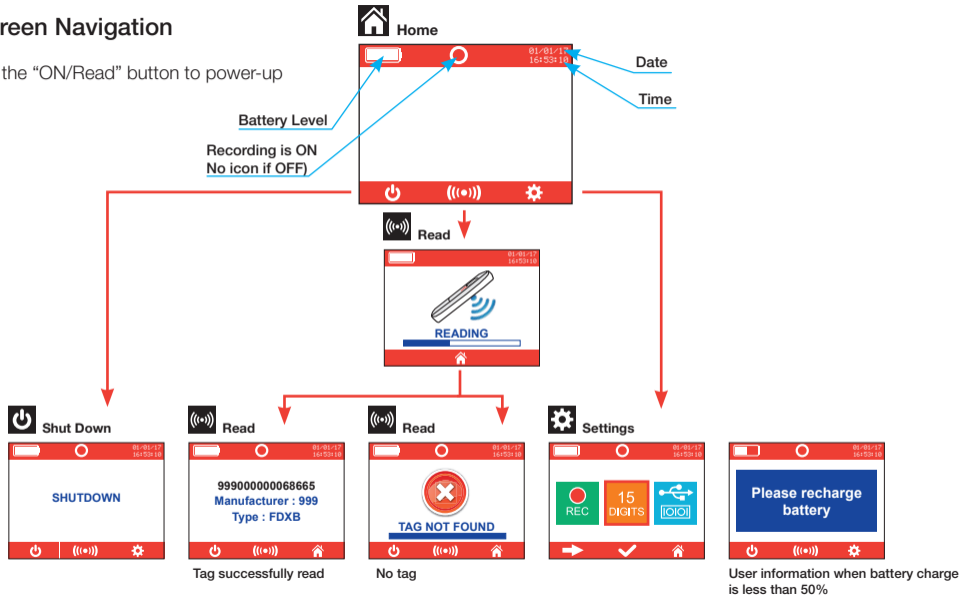- Press the "ON/Read" button to power-up

Connecting COMPACT MAX to a PC via the USB port

- Connect COMPACT MAX to your PC using the USB cable provided with the reader.
- To access the ID codes stored in the reader's memory via USB, the reader must be turned off. Up to 4'000 ID codes can be stored in reader. The COMPACT MAX memory is accessible thanks to S-ID companion SW, or in mass storage configuration. In this configuration, available both on PC and MAC, COMPACT MAX is seen like a standard USB stick. Codes are recorded in the "Session.txt" file, stored in a folder named CMAX and available on your PC or MAC.
- If the reader is turned ON while connected to a PC, the reader will behave as a PC peripheral and ID Codes will be sent to the PC via the USB cable at each reading.

Two interfaces are supported, as described in the "Icons Legend of the Settings menu":

Keyboard wedge: the reader is seen by the PC as a keyboard. The codes will be displayed on the PC screen in any window where a mouse cursor is active. No configuration on the PC is needed.

VCP: reader is connected to one of the Virtual Com Ports of the PC and sends the codes using RS232 protocol. Codes are received on a hyper-terminal or using Datamars S-ID SW.
(Requires a specific PC driver that can be downloaded from www.pet.datamars.com)

3 Using COMPACT MAX to Scan Microchips

Scanning a Microchip (fig. 1)

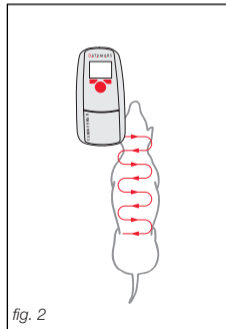
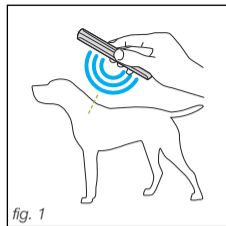
- Once the reader is switched on, hold it close to the animal and press the “ON/Read” button
- COMPACT MAX will emit an audible “beep” when the scanner successfully detects a microchip
- To stop scanning if no tag is found, press the “ON/Read” button once again

Effective Scanning (fig. 2)

- Hold the scanner close to, or touching the animal
- For optimal reading, keep the scanner parallel to the animal’s body rather than pointing it at the animal
- Scan slowly and repeatedly over the entire body to check whether or not a microchip is present
- Wave the scanner back and forth slowly while scanning, because microchips can be in various orientations within the animal
- Scan first in a horizontal, then a vertical “S” pattern down the animal’s body, as shown in the diagram beside
- Consider scanning each animal more than once to ensure any present microchip can be localized by the scanner

4 Battery

COMPACT MAX can be charged in two ways. You can connect the USB cable to the wall-mounted adapter or you can connect the USB cable to a USB port on your PC. To extend battery life, COMPACT MAX has an “Auto-off” function that automatically turns off the reader after 2 minutes (default value) of inactivity. As the device may discharge during storage, please make sure that the battery is charged regularly.

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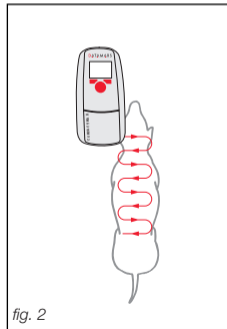
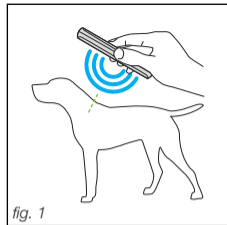
Distributore:

DATAMARS

DATAMARS SA, Switzerland
www.datamars.com

The warranty covers damage caused by faulty material or manufacturing defects for the period of 2 years from the date of purchase. Equipment repaired by a third party, as well as damage caused by the customer are not covered by this warranty. Claims for covering of damage exceeding the limits of the present warranty are not accepted. In case of repair or replacement the warranty is neither extended nor renewed. In the event of a claim under this warranty contact your dealer directly and present the warranty certificate stamped and dated by the dealer or the purchase invoice. Repairs carried out without a valid warranty certificate will be invoiced to the customer.

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