

COMPACT MAX+

Quick Start Guide


Datamars INC

345 West Cummings Park
Woburn, MA 01801, United States

Phone: +1 781 281 2216

Fax: +1 781 300 7330

DATAMARS
Pet

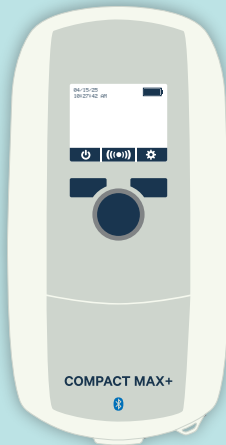


1. Before you Start

This manual is for COMPACT MAX+, universal portable scanner to identify pets.

Supported microchip technologies

FDX-B (ISO), FDX-A, Trovan and Avid

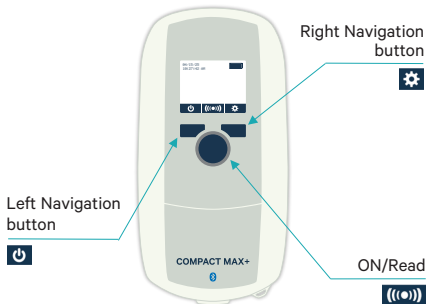

Icon Legend



Info (Firmware, Serial Number and bootloader versions)



BLE ON/OFF



Memory File enabled (recording) / Memory File disabled (no recording)



Download memory file on PC enabled (USB connected)
Download memory file on PC disabled



Delete memory content

Step 1

To start using your scanner it is recommended to fully charge the battery as the device may discharge during storage.

Step 2

Turn ON your scanner by clicking on the central blue button and start scanning microchips by using the same button.

Step 3

Download the “Datamars Pet Scanner Support” on your smartphone. The APP is available for free on the Google Play Store for Android devices or the Apple App Store for iOS devices.

Step 4

Open the APP and follow the pairing instructions.

The APP will allow you to:

- Update the scanner with the latest software version.
- Look up microchip numbers in integrated databases.
- Share the recorded microchip numbers.

- Synchronize date and time with your smartphone.
- Change the scanner settings.

Step 5

When the first synchronization is terminated, keep using your scanner to read microchip ID numbers. To learn more about your scanner capabilities, read through the full manual and explore the APP.



Scan to read
the complete
User Manual

pet.datamars.com

WARRANTY CARD

Model / Serial number:

DATAMARS
Pet

Limited warranty:

This warranty covers defects in materials and workmanship under normal use and in accordance with product instructions for a period of one (1) year from the date of original retail purchase.

It is understood that the warranty period is subject to applicable local laws.

Exclusions:

This warranty does not cover damage resulting from abuse, accident, alteration, misuse, improper installation, lack of reasonable care, unauthorized repair, or any issues not arising from defects in material or workmanship.

Claim Procedure:

To obtain warranty service, contact your authorized dealer and present this warranty card along with the original purchase invoice, otherwise the warranty service will not be provided.

Limitation

The manufacturer's sole obligation under this warranty is to repair or replace the product, or provide a refund, at its discretion.

Legal Rights

This warranty gives you specific legal rights; you may also have statutory rights which vary by jurisdiction.