# **COMPACT MAX+**

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#### 1. Before you start

This manual is for COMPACT MAX +, portable scanner to identify pets.

#### 2. To get started

#### Step 1

To start using your reader it is recommended to fully charge the battery, as the device may discharge during storage.

#### Step 2

Turn ON your reader by clicking on the central blue button and start scanning microchips by clicking on the same button.

Additional Steps for an enhanced scanning experience

#### Step 3

Download the "Datamars Pet Scanner Support" app on your smartphone. The app is available for free on the Google Play Store for Android devices and the Apple APP Store for iOS devices.

#### Step 4

Open the App and follow the set-up and pairing instructions.

The app will allow you to:

- Update the reader with the newer software version
- Look up microchip numbers in PetMaxx
- Share the recorded microchip numbers
- Synchronize date and time with your smartphone
- Change reader settings

## Step 5

When the first synchronization is completed, keep using your scanner to read microchip ID numbers. To learn more about your reader capabilities read through the full user manual available on the Datamars Pet website and explore the APP.

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